

2018.06.11

www.rejuvenation.com

INSTALLATION, OPERATING AND CARE INSTRUCTIONS:

SAFETY WARNINGS:

- DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these Warnings and Instructions, contact a health care professional, dealer or technical personnel for assistance before attempting to install this equipment - otherwise, injury may occur.
- After ANY adjustments, repair or service and BEFORE use, make sure that all attaching hardware is tightened securely.
- Grab Bars add security for bath and commode areas if attached as instructed. Location, size, and angle of bars should be determined by a qualified professional to provide optimum safety and function.
- Use EXTREME caution on wet tub or floor surfaces.
- Users with limited physical capabilities should be supervised or assisted in bath and commode areas, even when using the Grab Bars.
- This Grab Bar can provide support and increased stability for an individual weighing up to 500 pounds. Grab Bars are NOT designed to support the total weight of an individual. Use the bar for assistance ONLY.
- Be sure Grab Bars are correctly and securely installed. NEVER use Grab Bars if any looseness is noted.

TOOLS REQUIRED:

- 3/8-inch Power Drill
- 7/64-inch Drill Bit (to drill pilot hole for installation of wood screw into wood wall stud)
- 1/4-inch Carbide Tip Drill Bit (if drilling through ceramic tile)
- #2 Phillips Screwdriver
- Safety Glasses
- Stud Finder

PARTS SUPPLIED:

Two 2() #14 x 3-1/2 inch wood screws

WARNING

Grab Bar May Not Provide Designed Amount of Support Unless these Installation Instructions are Strictly Followed.

Grab bar may be installed vertically, horizontally or at an angle, so long as mounting screws on both ends of the Grab Bar are secured into a wall stud (see Figure 2). Locate the wall studs behind the wall before beginning installation (a stud finder can be obtained from a hardware store). Additional framing may be required.

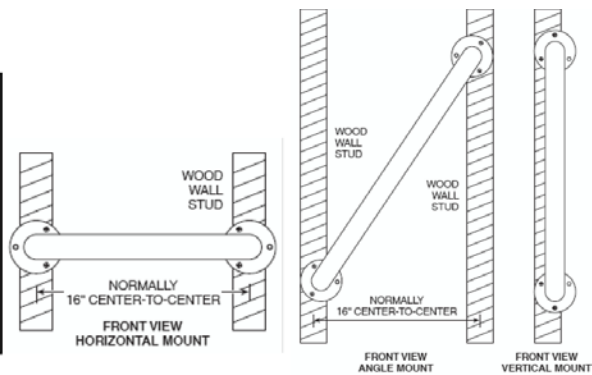


Figure 2

